

Validation of the RealStar® Dengue Type RT-PCR Kit 1.0 for Differentiation of Dengue Virus Serotypes 1–4.

Hussein El Halas, Nils Bunte, Mareen Zaruba, Markus Hess and Stephan Ölschläger
altona Diagnostics GmbH, Research and Development, Mörkenstr. 12, 22767 Hamburg, Germany

Introduction

Dengue virus (DENV; family *Flaviviridae*) is the causative agent of dengue fever. The virus is transmitted mainly by *Aedes* mosquitoes. Dengue infections can range from a mild flu-like syndrome to a severe dengue hemorrhagic fever (DHF) or Dengue shock syndrome (DSS). Dengue infections are a permanent problem in tropical and sub-tropical countries. The spread of dengue has increased dramatically around the world in recent decades and a large worldwide outbreak took place in 2015 [1]. There are four Dengue serotypes that can cause Dengue infections in humans (DENV-1, DENV-2, DENV-3 and DENV-4). The first infection leads to a long-lasting protection to that serotype. However, a second infection with a different serotype leads to a higher risk of developing dengue hemorrhagic fever [2]. For this reason, it is important to have a diagnostic test that differentiates between the four different Dengue virus serotypes.

Materials and Methods

altona Diagnostics GmbH developed the RealStar® Dengue Type RT-PCR Kit 1.0 as a second-line diagnostic assay for identification of the four known Dengue virus serotypes in known Dengue-positive samples.

The limit of detection was determined using probit analysis after testing replicates of limited dilutions of quantified *in vitro* transcribed RNA containing the RT-PCR target sequence. The limit of detection was calculated for each of the four different Dengue virus serotypes.

The ability of the RealStar® Dengue Type RT-PCR kit 1.0 assay to differentiate between the four different Dengue virus types was evaluated by testing the 2013 and 2016 QCMD Dengue panels.

Analytical Sensitivity

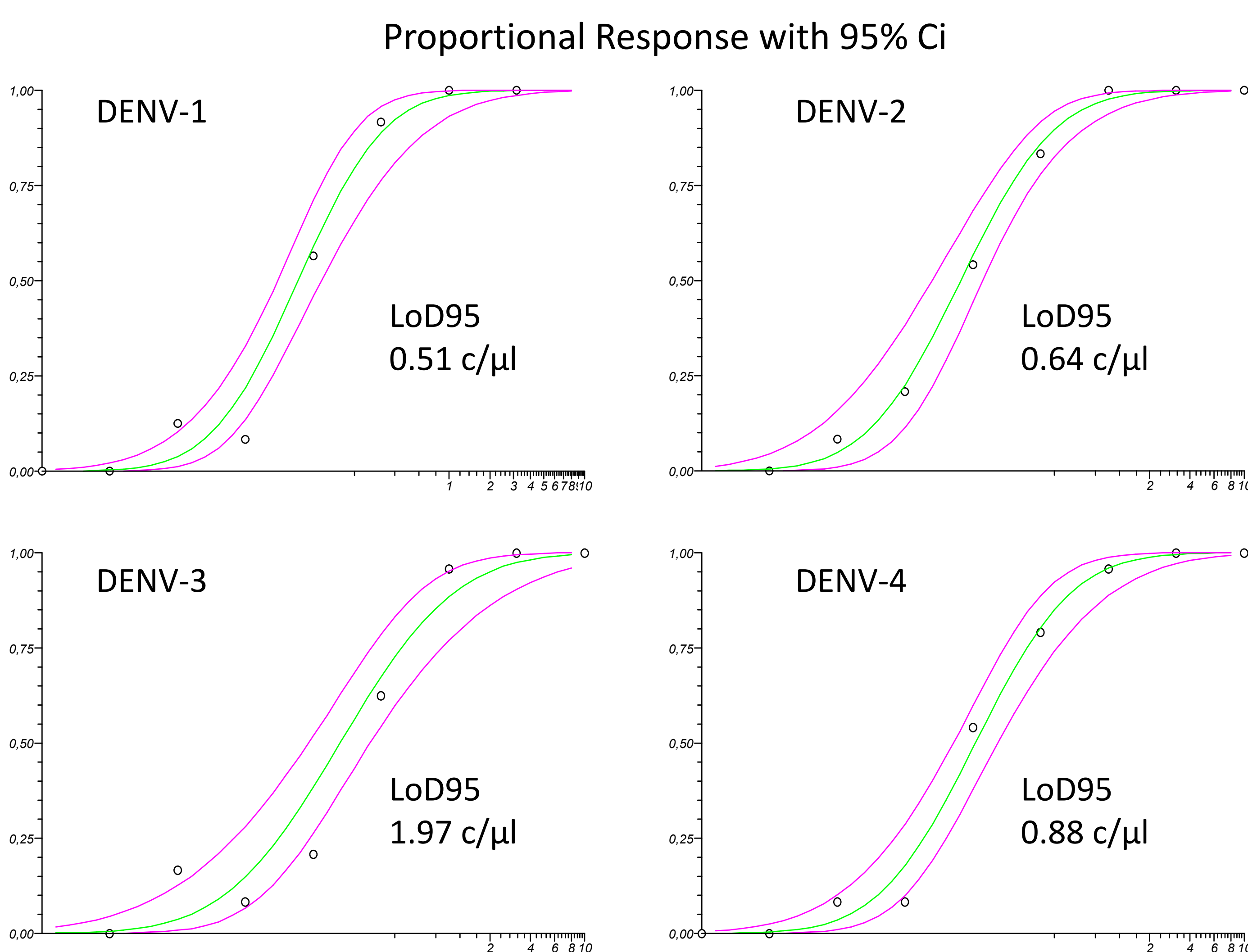


Figure 1
Probit analysis for the RealStar® Dengue Type RT-PCR Kit 1.0.

The analytical sensitivity (95% Limit of Detection, LoD95) was determined for each type by testing replicates of half-logarithmic dilutions of virus specific *in vitro* transcribed RNA. The X-axis shows the concentration of RNA and the Y-axis the proportion of positive results. For DENV-1 the LoD95 is 0.51 c/µl (confidence interval (CI) 0.31 to 1.15); for DENV-2 0.64 c/µl (CI 0.37 to 1.47); for DENV-3 1.97 c/µl (CI 0.45 to 11.68); for DENV-4 0.88 c/µl (CI 0.51 to 2.03).

Diagnostic validation

Samples from the 2013 and 2016 QCMD Dengue panels were tested with the RealStar® Dengue RT-PCR Kit 2.0. Positive samples were subsequently typed with the RealStar® Dengue Type RT-PCR Kit 1.0. The observed results were 100% correct for both panels.

Table 1: Dengue QCMD 2013 Results

Sample ID	RealStar® Dengue RT-PCR Kit 2.0 [Ct-value]	RealStar® Dengue Type RT-PCR Kit 1.0 [Ct-value / serotype]	QCMD Sample Content
DENVRNA13-01	26.81	21.55 / DENV-1	Dengue virus Type 1
DENVRNA13-02	29.16	27.32 / DENV-1	Dengue virus Type 1
DENVRNA13-03	29.45	24.74 / DENV-1	Dengue virus Type 1
DENVRNA13-04	34.65	28.33 / DENV-1	Dengue virus Type 1
DENVRNA13-05	34.78	31.21 / DENV-1	Dengue virus Type 1
DENVRNA13-07	35.17	36.29 / DENV-2	Dengue virus Type 2
DENVRNA13-08	32.11	26.60 / DENV-3	Dengue virus Type 3
DENVRNA13-09	35.07	30.11 / DENV-3	Dengue virus Type 3
DENVRNA13-10	33.92	25.86 / DENV-4	Dengue virus Type 4
DENVRNA13-11	negative	Not tested	Non Dengue virus flaviviruses
DENVRNA13-12	negative	Not tested	DENV Negative VTM

Table 2: Dengue QCMD 2016 Results

Sample ID	RealStar® Dengue RT-PCR Kit 2.0 [Ct-value]	RealStar® Dengue Type RT-PCR Kit 1.0 [Ct-value / serotype]	QCMD Sample Content
DENVRNA16-01	27.83	21.66 / DENV-1	Dengue virus Type 1 (core)
DENVRNA16-02	36.35	31.27 / DENV-1	Dengue virus Type 1 (educational)
DENVRNA16-03	35.33	33.74 / DENV-2	Dengue virus Type 2 (educational)
DENVRNA16-04	32.62	31.19 / DENV-2	Dengue virus Type 2 (core)
DENVRNA16-05	31.4	25.42 / DENV-3	Dengue virus Type 3 (core)
DENVRNA16-06	negative	Not tested	Dengue virus negative (core)
DENVRNA16-07	30.05	26.03 / DENV-4	Dengue virus Type 4 (core)
DENVRNA16-08	30.85	24.76 / DENV-1	Dengue virus Type 1 (core)
DENVRNA16-09	34.17	28.05 / DENV-1	Dengue virus Type 1 (core)
DENVRNA16-10	negative	Not tested	Non dengue virus flaviviruses (core)

Figure 2: Amplification curves of QCMD panel samples with the different Dengue virus Type assays contained in the RealStar® Dengue Type RT-PCR Kit 1.0. Samples were run on a CFX96™ Real-Time System (Bio-Rad). Fluorescence signals of DENV-1 and DENV-3 assays are measured in the FAM channel and the signals of the DENV-2 and DENV-4 assays in the Cy5 channel.

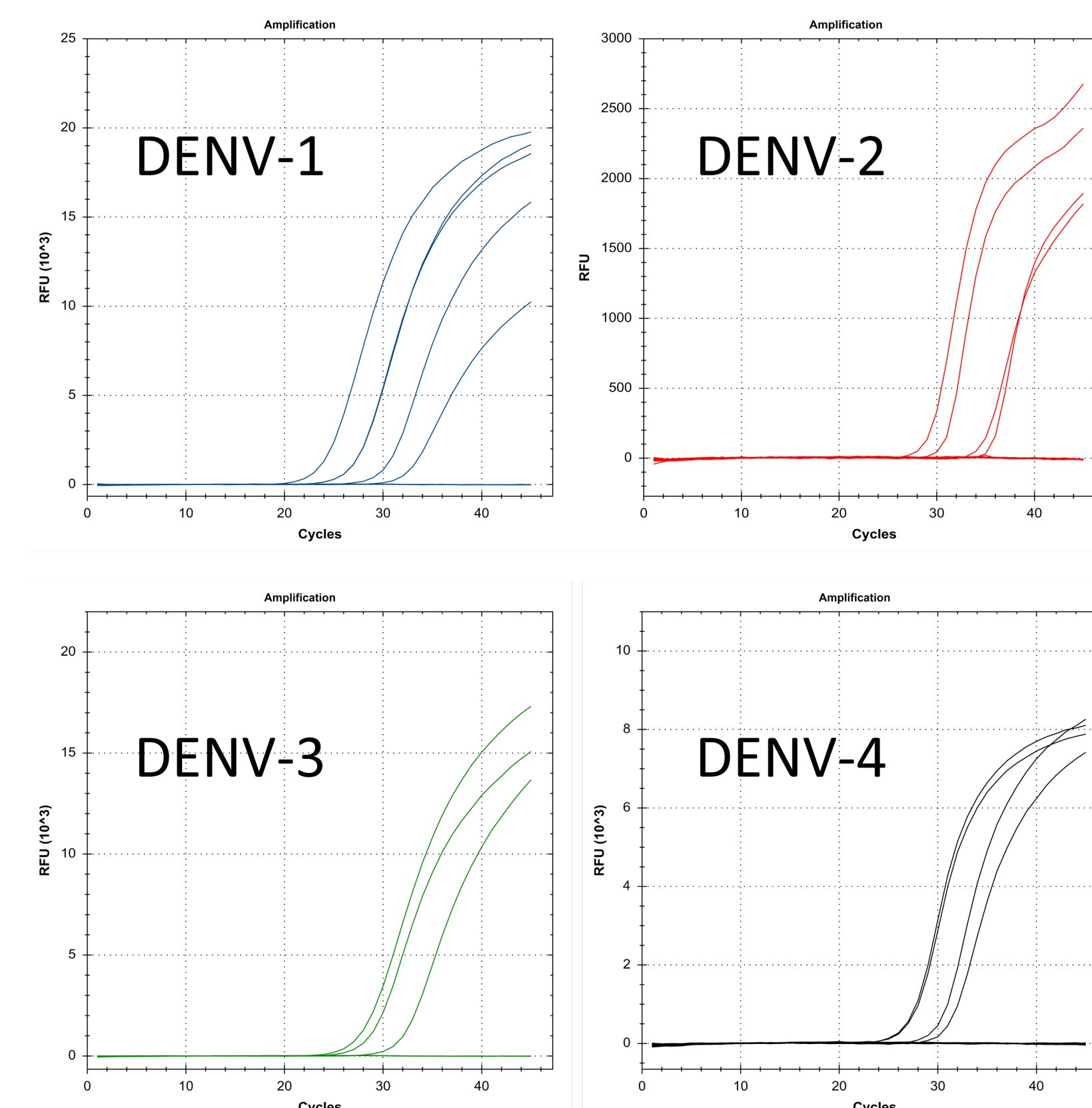


Table 3: Reactivity and cross-reactivity of the Dengue assays 1-4 contained in the RealStar® Dengue Type Kit.

Virus RNA	RealStar® Dengue Type RT-PCR 1.0 assay			
	DENV-1	DENV-2	DENV-3	DENV-4
Dengue virus 1	+	-	(+)*	-
Dengue virus 2	-	+	-	-
Dengue virus 3	-	-	+	-
Dengue virus 4	-	-	-	+

* weak cross-reactivity of the Dengue virus 3 assay with high RNA titers of Dengue virus serotype 1

Conclusion

The RealStar® Dengue Type RT-PCR Kit 1.0 enables the user to distinguish successfully between the four different Dengue virus serotypes in Dengue virus positive samples.

References

- [1] WHO Fact Sheets. 2016. *Dengue and severe Dengue*. [ONLINE] Available at: <http://www.who.int/mediacentre/factsheets/fs117/en/>. [Accessed 10 October 2016].
- [2] Rico-Hesse, R. (2009). Dengue virus markers of virulence and pathogenicity. *Future Virology*, 4(6), 581-589.

Contact

stephan.oelschlaeger@altona-diagnostics.com

altona Diagnostics GmbH
Mörkenstr. 12
22767 Hamburg
Germany

